



BTM921 Body Thermometer

Quick Start Guide

1

Introduction





The body thermometer BTM921 is used to control the electrical doors in your home. You can simply insert the body thermometer into the existing and use the cable to integrate it with your existing switch. This easy-to-install smart device allows you to control your lights remotely.

This guide will provide you with an overview of the product and help you get through the initial setup to installation.

2

Get to know your device

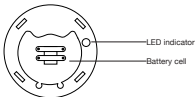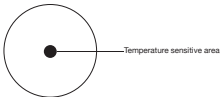

3

Get started

3.1 To get started, you will need:

A Bluetooth gateway.

The SmartOWON mobile app.

3.2 Join the thermometer to the Bluetooth gateway

1. Install the battery.
2. Set you Bluetooth gateway to permit joining,
3. Follow the instructions in the mobile app to join the thermometer to the gateway.

3.3 Wearing

Please refer to the instructions below to install the body thermometer.

1. Take one of sleeve off, and stick the body thermometer under armpits.
2. Put down the dress without wearing the sleeve.

Note:

1. Adult should help child to wear and operate the device.
2. Keep your arm tightly at least 5 mins.
3. Put the mobile phone and the thermometer on the same side closely.

4

Technical Specifications

Wireless Connectivity	• BLE 4.0
-----------------------	-----------

Battery	• CR2016
---------	----------

Working Environment	• 10-60°C
---------------------	-----------

Accuracy	• 0.1°C
----------	---------

Measurement	• 32-44°C
-------------	-----------

Possible Error	• $\leq 0.12^{\circ}\text{C}$
----------------	-------------------------------

Dimensions	• R30.5*8.5 mm
------------	----------------

Weight	• 93g
--------	-------

5

Safety Handling

WARNING: Failure to follow these safety notices could result in fire, electric shock, other injuries, or damage to the Body thermometer and other property. Read all the safety notices below before using the Body thermometer.

- Avoid high humidity or extreme temperatures.
- Avoid long exposure to direct sunlight or strong ultraviolet light.
- Do not drop or expose the unit to intense vibration.
- Do not disassemble or try to repair the unit on your own.
- Do not expose the unit or its accessories to flammable liquids, gases or other explosives.